# BORT Codyn Sitzkissen

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**REF** 190 530

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D19053012021-101003 ML | Rev.01

### DE BORT Codyn Sitzkissen

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### EN BORT Codyn Seat Cushion

Many thanks for placing your trust in a medical device from BORT GmbH. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this medical device

#### Intended purpose

This medical device is an air-filled seat cushion for relieving and positioning the sacral region without pressure.

#### Indications

Coccygodynia, condition after coccyx contusion and fracture, perianal relief in case of haemorrhoids, after rectal surgery, perianal abscesses, condition after episiotomy.

#### Contraindications

Impaired sensation and circulation in the specific body region, skin diseases in the specific part of the body.

#### Application risks/Important notes

This medical device is a prescribable product. Discuss the use and duration with your treating physician. The expert staff from whom you have received the medical device will instruct you regarding its use.

- remove the medical device prior to radiological examinations
- in case of numbness, release air or do not use the aid
- in case of persistent complaints, consult the physician or specialist retailer
- use the medical device in accordance with therapeutic needs
- only use other products simultaneously after consultation with your physician
- do not make any changes to the product
- do not wear it on open wounds
- do not use in case of intolerance of one of the materials used
- no re-use – this medical aid is intended for treating one patient
- whilst wearing the seat cushion, please neither use any creams nor ointments on or around it as they can destroy the material
- do not use if the patient's weight exceeds 150 kg

#### Fitting instructions

Place the inflated cushion on a chair with the printed BORT logo in the direction of the backrest. The filling quantity should be chosen to guarantee pressure relief for the coccyx. This can vary according to the patients' weight.

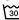
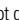
#### Removal

Store the seat cushion at room temperature.

#### Material composition

50% polyamide (PA), 50% polyurethane (PU)

#### Cleaning information

 Delicate wash 30 °C  Do not bleach  Do not dry in a tumble dryer  Do not iron  Do not clean chemically  
Do not use fabric conditioner. Stretch back into shape and dry in the fresh air.

#### Guarantee

The legal regulations of the country in which you acquired the product apply to the purchased product. Please contact your specialist retailer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been properly observed, the warranty may be impaired or cancelled. The warranty does not cover use of the product inappropriate for the indication, non-observance of application risks, instructions and unauthorised modifications to the product.

#### Useful life/Lifetime of the product

The lifetime of the medical device is determined by its natural wear and tear if treated appropriately and as recommended.

#### Duty of notification

If a serious worsening of the state of health occurs when using the medical device, you are obliged to notify the specialist dealer or us as the manufacturer and the Medicines & Healthcare products Regulatory Agency (MHRA).

You can find our contact information in these instructions for use. You can find the contact information for the appointed authority for your country under the following link: [www.bort.com/md-eu-contact](http://www.bort.com/md-eu-contact).

#### Disposal

Upon the termination of use, the product must be disposed of in accordance with the corresponding local requirements.

#### Declaration of conformity

We confirm that this device conforms with the requirements of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL. You can find the current declaration of conformity under the following link: [www.bort.com/conformity](http://www.bort.com/conformity)

Status: 10.2021

Medical device |  Single patient – multiple use

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