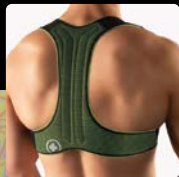
BORT Geradehalter Sport

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
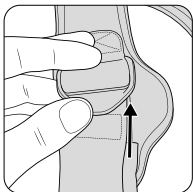
Sport

Illustrationen
Figures



1



2

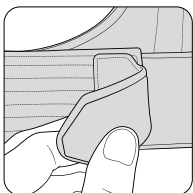3

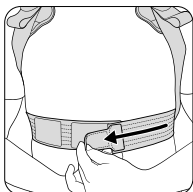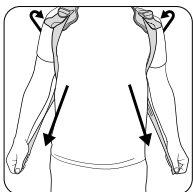4

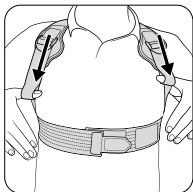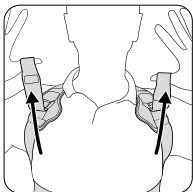Sprachen Languages

DE	deutsch	Gebrauchsanweisung	04
EN	english	Instructions for use	08
FR	français	Mode d'emploi	12
ES	español	Instrucciones de uso	16
IT	italiano	Istruzioni per l'uso	20
NL	nederlands	Gebruiksaanwijzing	24
CS	český	Návod k použití	28
ET	eesti	Kasutusjuhend	32
PL	polski	Instrukcja użytkowania	36
RO	românesc	Instrucțiunile de utilizare	40

PDF: ga.bort.com

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Many thanks for placing your trust in a medical device from BORT GmbH. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this medical device.

Intended purpose

This device is a brace for straightening and relieving the thoracic spine.

Indications

Straightening and relief of the thoracic spine, e.g. misalignment of the thoracic spine, dorsalgia (pain in the thoracic spine region)

Contraindications

Thrombosis risk, extreme varicosis, peripheral arterial occlusive disease (PAOD), lymph drainage disorders, also unclear soft tissue swellings distal to the aid positioned, sensory and circulatory disorders of the region of the body treated, skin diseases in the part of the body treated, latex allergy.

Application risks/Important notes ⚠

This medical device is a prescribable product. Discuss the use and duration with your treating physician. The expert staff from whom you have received the medical device will select the appropriate size and instruct you regarding its use.

- remove the medical device prior to radiological examinations
- remove it during longer periods of rest, e.g. when sleeping
- in case of numbness, loosen the medical device and remove it if necessary
- in case of persistent complaints, consult the physician or specialist retailer
- use the medical device in accordance with therapeutic needs
- only use other products simultaneously after consultation with your physician
- do not make any changes to the product
- do not wear it on open wounds
- do not use in case of intolerance of one of the materials used
- no re-use – this medical aid is intended for treating one patient
- whilst wearing the brace, please neither use any creams nor ointments on or around it as they can destroy the material.

Fitting instructions

- 1** Loosen the Velcro which fixes the thumb loop on the restraint and pull the loop upwards to the rucksack buckle.
- 2** Open the chest strap using the Velcro fastener.
- 3** Guide the brace around the shoulders like a rucksack. Pull the brace over the chest straps and close the Velcro below the chest.
- 4** Then, actively straighten your thoracic spine, guide your thumbs through the thumb loops and release the Velcro by guiding the loops upwards. Following this, pull the restraints downwards tightly and fix the strap on the course of the strap using the Velcro.

Removal

Remove the thumb loop from the Velcro and guide it to the rucksack buckle, thus loosening the restraint guidance. Open the chest strap and slip the corset off over the back as for a rucksack.

Material composition

Polyamide (PA), polyester (PES), elastane (EL)

Cleaning information

 Delicate wash 30°C  Do not bleach  Do not dry in a tumble dryer
 Do not iron  Do not clean chemically

Information: Before cleaning, pay attention to remove the splints in the back plate in order to avoid damages to the brace and your washing machine. Close the Velcro fastening to avoid damage to other laundry items. Do not use fabric conditioner. Stretch back into shape and dry in the fresh air.

Guarantee

The legal regulations of the country in which you acquired the product apply to the purchased product. Please contact your specialist retailer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been properly observed, the warranty may be impaired or cancelled. The warranty does not cover use of the product inappropriate for the indication, non-observance of application risks, instructions and unauthorised modifications to the product.

Useful life/Lifetime of the product

The lifetime of the medical device is determined by its natural wear and tear if treated appropriately and as recommended.

Duty of notification

If a serious worsening of the state of health occurs when using the medical device, you are obliged to notify the specialist dealer or us as the manufacturer and the Medicines & Healthcare products Regulatory Agency (MHRA).

You can find our contact information in these instructions for use. You can find the contact information for the appointed authority for your country under the following link: www.bort.com/md-eu-contact.

Disposal

Upon the termination of use, the product must be disposed of in accordance with the corresponding local requirements.

Declaration of conformity

We confirm that this device conforms with the requirements of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL.

You can find the current declaration of conformity under the following link:
www.bort.com/conformity

Status: 10.2021

Medical device |  Single patient – multiple use

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