DOMICILE – HOMECARE

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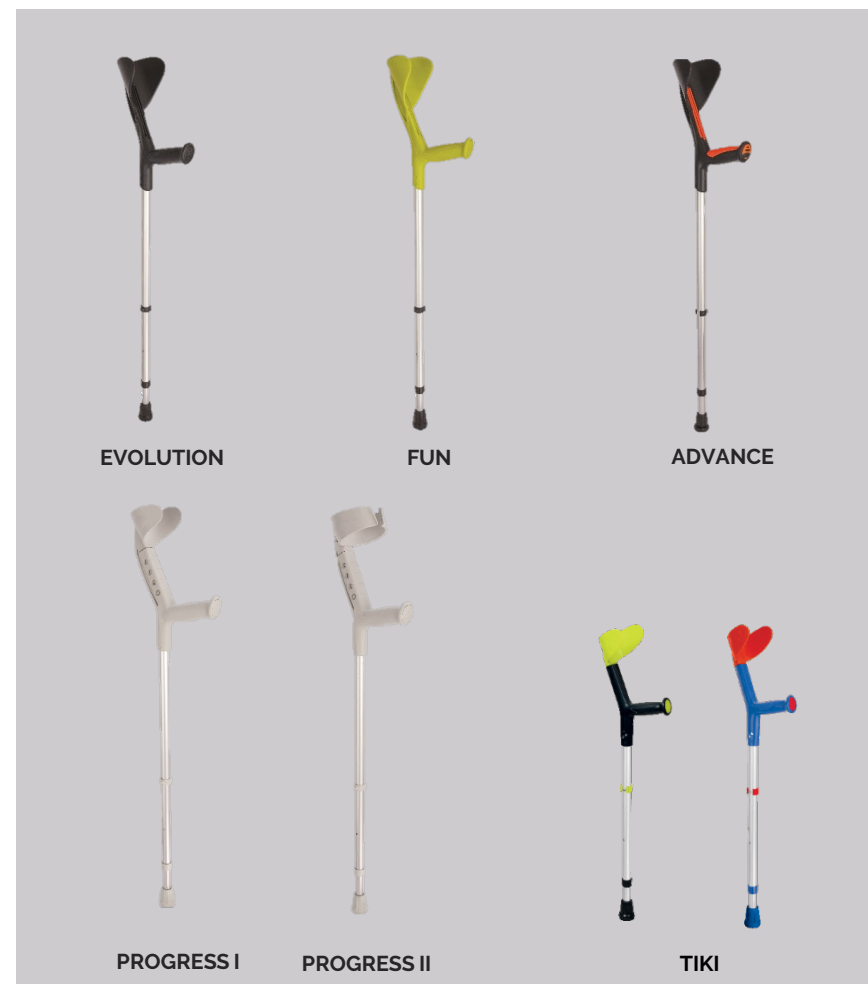
2025-01-31

IMPROVE THE NURSING LIFE*

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EN – Instructions for installation and use

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Copyright 2025 par HERDEGEN SAS– ALL RIGHTS RESERVED

In the interests of continuous improvement, HERDEGEN SAS reserves the right to change any detail or specification in this document without prior notice.

The information contained in this document is confidential. It may only be shared with a third party with the prior written consent of HERDEGEN SAS. This document may not be reproduced or communicated, in full or in part, in any way or by any means, whether technical or electronic, by photocopying, recording, or using any information or data extraction systems.

You can obtain a new copy of this document from HERDEGEN SAS by contacting your regional sales representative or directly contacting HERDEGEN SAS.

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FOREWORD

You have just purchased a HACARE product and we are pleased to have you as one of our clients. We also thank you for placing your trust in our company and our products.

Your trust assures us that the HACARE company continues to meet your specific requirements and needs. That's why we always strive to innovate, providing you with quality solutions and products that improve the comfort and quality of patient care while making the caregiver's daily tasks easier.

HACARE products have gained the confidence of many professionals and institutions due to their functional and aesthetic qualities, and are now considered to be a solid benchmark. Simplicity, reliability and safety are the hallmarks of our electrical healthcare cradles.

You've chosen HACARE, and we're committed to satisfying your needs.

However, you are responsible for the proper use of the cradles. That's why we invite you to familiarize yourself with the technical specifications, and the instructions for use and maintenance of the equipment by reading through this manual.

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4. NETTOYAGE, DESINFECTION, MAINTENANCE

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EN - Please read these notes carefully before use. Make sure to follow the instructions and keep them in a safe place. If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

1. SAFETY INDICATION, ABNORMAL USES, DANGERS

- ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- Don't store outside and don't let under the sun (Beware: don't let the product in a car in case of high outside temperature).
- Don't use in extreme temperature (above 38°C or 100°F or below -15°C or 5°F).
- Check carefully all parts of this item (screws, tubes, tips, grips...).
- Check regularly if all parts are tightened (clips, handle, rubber tip...).
- Hold firmly and completely the handle when using.
- In case of falling : let the product fall down on the side, so that you don't fall on it.
- Always move forward quietly and with short steps to ensure a good balance of your weight and to avoid losing your balance.
- Don't use on wet or icy surfaces. (snow, ice, water...)
- Don't use in stairs, severe risk of injuries
- Use only as a walking aid and in proper way : don't use as a lever or to move furniture.

- ALWAYS observe the weight limit on the labelling of your walker.
- No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ALWAYS test to see that the walking aid and attachments are properly and securely locked in place before using.
- DO NOT hang anything on the Walking aid.
- In case you didn't use your walking aid for a long time, let it checked by the technical staff, before using.
- Be careful when using: beware holes and deformations in the floor.
- The suction cap must not be too worn down. If the cap shows considerable wear, ask your supplier or nursing staff to replace it (photo A).

- Any serious incident that has occurred in relation to the device shall be reported to the manufacturer (qualite@hacare.com) and the competent authority of the member state in which the user and/or the patient is established.

- The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts. Likewise, the warranty excludes parts or working parts subject to natural wear and tear.

2. INTENDED USE, INDICATIONS AND CONTRAINDICATIONS

Walking aids are used for two purposes: as part of a rehabilitation programme when the user is recovering from an injury or operation and as a long-term aid to mobility when the user has a permanent difficulty with walking. These devices help in maintaining balance, aid in the prevention of injuries (particularly falls), and allow the person to attend to his/her errands independently. Mobility aids can be used indoor or outdoor, depending on the model. Mobility aids are used to modify the weight redistribution: some of the weight carried through the legs when walking is transferred through the arms of the frame or stick as it is leant on for support.

Indications:

- Loss of autonomy due to walking disorders
- Balance and/or posture disorders, walking problems
- Rehabilitation programme after an injury or an operation

Contraindications:

- Severe cognitive disorder, severe loss of balance, severe motor or perception disorders or weakness in the upper limbs
- Patient weight is greater than the product's supported weight.
- Other contraindications: to be determined by the prescribing physician based on the patient's condition.

3. INSTALLATION

Only specialists can adjust the product and teach you how to use it. A wrong use or adjustment could lead to overstress of the device or of the patient. The device must be adjusted individually staff to suit your needs. Ask the technical staff how to use this product and make sure that you understood clearly their explanation. In case of doubt, don't hesitate to ask again.

Height Adjustment : Take out the clip from the tubes using both thumbs, then adjust the interior pipe to the desired height and lock again with the clip. Before using the crutch, check that the metal pin of the clip has gone through both pipes and exits from the outer pipe. The crutches have 2 adjusting holes on the outside pipe and 5 holes on the inside pipe, what means 10 adjusting positions. The crutch can be adjusted every 2.5 cm.

Keep the crutch close to your body while going to avoid accidents.

4. CLEANING AND DISINFECTION

The crutch has to be cleaned regularly with a mild cleaning agent and a soft cloth. Never use oily substances (slip hazard). Ask your retailer or the technical staff to check the walking aid regularly.

The rubber tips must be checked and changed regularly.

5. DISPOSAL

Dispose of this product in accordance with your regional waste disposal regulations.

6. TECHNICAL SPECIFICATIONS

See the table above. Replace the ferrule when the wear indicator (see: [A]) indicates it. Biocompatible. The warranty duration of the product and its various elements is specified in the commercial conditions.

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